


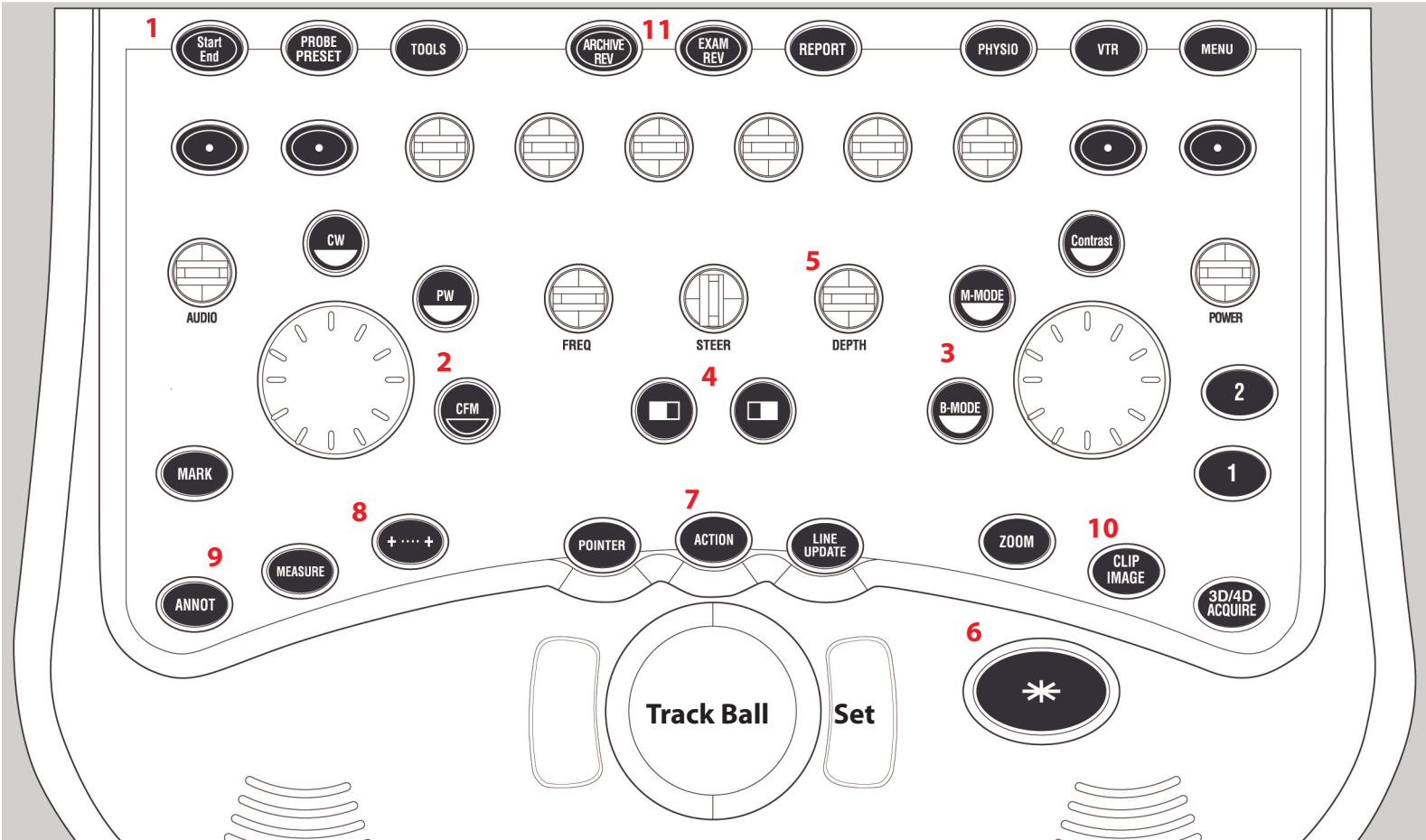

MyLab Quick Scan Card

1. **Start/End Exam**
2. **Doppler Controls**
 - a. CFM = Color Doppler
 - b. PW = Pulse Wave
 - c. Dial - increase / decrease doppler gain
3. **B-Mode Controls**
 - a. Push B-Mode to exit Doppler modes
 - b. Dial - increase / decrease 2D gain
4. **Split Screen**
 - a. Left button activates dual imaging
 - b. Right button activates 2nd image
 - c. Press freeze button to freeze
 - d. Action button activates toggle between image screens
 - e. Press Annot (7) to annotate
5. **Depth adjustment**
6. **Freeze image**
7. **Action Key** allows you to select text and move
8. **Caliper button**
 - a. Measures diameters in 2D
 - b. Measures time on Spectral waveform
 - c. Press "Set" key to release second (multiple) calipers
9. **Annot**
 - a. Opens **Annotation library**
 - b. Scroll with track ball to highlight text
 - c. Use trackball to position text
 - d. Press "Set" to lock position
10. **Clip/Image**
 - a. Press to save image
11. **Archive/Exam Review**
 - a. Archive Review used to export exams
 - b. Exam Review used to access patient exam measurements



United Medical Instruments

Toll Free: 877.490.7036 • Phone: 408.278.9300 • Fax: 408.278.9797

External Storage of Patient Exams

Images and Clips can be reloaded for each patient and a specific exam can be reviewed. Specific measurements can be taken and saved on the reloaded images.



1. Press the Archive Rev button.
2. Press the set key to highlight the patient's exam.
3. Next, move the cursor to the last name that you want to export and let it hover over it. Press Shift and Set buttons to highlight all names between the two exams.
4. Move cursor with track ball to the **Export** menu and press set.



5. Select **CD/DVD** export method of choice in the drop down menu.
6. Move the cursor to "OK" and press the set key.

