

Terason™

Terason uSmart3300 Keyboard Functions		2/6/15
V 5.7.0 Software		Page 1 of 4

Various functions of the uSmart3300 can be invoked by the following keystrokes. The Remote Access section documents Ctrl key combinations intended to enable operation without using the uSmart3300 console, e.g. from a remote access session.

Key Combinations	Command
Alphanumeric Keys	Quick Text
Function Keys	
Esc	Escape Full Screen
Esc	Cancel Arbitration
F1	Help Finder
F2	Freeze/Live
F3	Save Frame
F4	Save Loop
F5	ECG Window
F6	Stress Dialog
F7	Needle Visualizer
F10	Full Screen
F12	Print Direct
Pause	Pause/Play
Key Pads	
Page Up	Freeze/Live
Page Dn	Acquire Frame

Terason™

Terason uSmart3300 Keyboard Functions		2/6/15
V 5.7.0 Software		Page 2 of 4

Ctrl+Left Arrow	Decrease TGC Gain
Ctrl+ Right Arrow	Increase TGC Gain
Ctrl+Up Arrow	Select Previous TGC Ctrl
Ctrl+Dn Arrow	Select Next TGC Ctrl
Remote Access	
Ctrl+1	Softkey 1
Ctrl+2	Softkey 2
Ctrl+3	Softkey 3
Ctrl+4	Softkey 4
Ctrl+Q	Softkey 5
Ctrl+W	Softkey 6
Ctrl+E	Softkey 7
Ctrl+R	Softkey 8
Ctrl+5	Previous Softkey Page
Ctrl+6	Select
Ctrl+7	Next Softkey Page
Ctrl+8	Depth Decrease
Ctrl+9	Focus Decrease
Ctrl+0	Split Left
Ctrl+-	Zoom In
Ctrl+=	Gain Increase
Ctrl+l	Depth Increase
Ctrl+O	Focus Increase

Terason™

Terason uSmart3300 Keyboard Functions		2/6/15
V 5.7.0 Software		Page 3 of 4

Ctrl+P	Split Right
Ctrl+[Zoom Out
Ctrl+]	Gain Decrease
Ctrl + \	Active Mode
Ctrl+Shift+8	Baseline Up
Ctrl+Shift+9	Scale Up
Ctrl+Shift+0	Steer Left
Ctrl+Shift+l	Baseline Down
Ctrl+Shift+O	Scale Down
Ctrl+Shift+P	Steer Right
Ctrl+F	Patient
Ctrl+G	Preset
Ctrl+H	Review
Ctrl+J	Report
Ctrl+K	EndStudy
Ctrl+L	Probe
Ctrl+Shift+S	Setup
Ctrl + ,	Calcs
Ctrl + .	Caliper
Ctrl + /	Cursor
Ctrl + ;	Body Marker
Ctrl + ’	Optimize
Ctrl+A	CW
Ctrl+B	2D

Terason™

Terason uSmart3300 Keyboard Functions

2/6/15

V 5.7.0 Software

Page 4 of 4

Ctrl+Shift+C	Color
Ctrl+D	PW
Ctrl+M	M-Mode
Ctrl+S	Tissue Doppler
Ctrl+T	Text
Ctrl+U	PW Update
Ctrl+Shift+X	Clear
Ctrl+Z	ZoomBox/Unzoom