

Entering Patient Information



1. Press patient button to enter patient information.
 - a. You must input a patient name to save/store an image.
 - b. If this is an emergency exam, you should enter at least the patients last name - you can edit patient information at the end of the exam.
2. Select Exam Type - Dialysis Access - Bottom of patient screen
3. Press Accept to start the exam - Soft menu button, bottom of the monitor



Performing Dialysis Access Exam



- 2.
- 3.
- 4.
5. & 6.
- 7.

1. Scan patient
2. Press PW button, use trackball to position the cursor and get a sample in the fistula
3. Press Freeze button
4. Use the left toggle switches to measure the top and bottom vessel walls
5. Use the trackball to measure the spectral waveform
6. Press the Enter button on the left side of the trackball to keep the measurement
7. Press Store button to save the image

Editing Emergency Patient Information

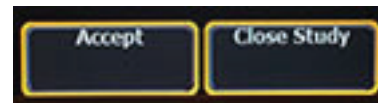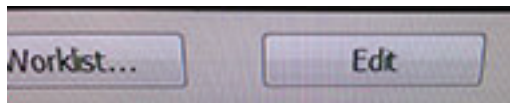

- 1.
- 2.



- 4.

You can edit the information input in the patient data screen:

1. Press Patient button
2. Press Edit
3. Enter all the correct information
4. Press Accept

Information on the patient exam images will be updated