


## Entering Patient Information



1. Press patient button to enter patient information.
  - a. You must input a patient name to save/store an image.
  - b. If this is an emergency exam, you should enter at least the patients last name - you can edit patient information at the end of the exam.
2. Select Exam Type - Dialysis Access - Bottom of patient screen
3. Press Accept to start the exam - Soft menu button, bottom of the monitor

## Performing Extremity Vein Mapping Exam

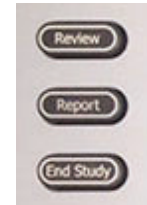


- 2.
- 3.
- 4.
- 5.
- 6.
- 7.
- 8.
- 9.

1. Scan patient
2. Press the Freeze button
3. Press the Calc button - use trackball to highlight measurement in popup window
4. Press the left Enter button to release selected measurement caliper
5. Use the trackball to position the 1st caliper
6. Press the left Enter button to release the 2nd caliper
7. Use the trackball to position the 2nd caliper
8. Press Store button to save the image with measurement - REPEAT UNTIL PROTOCOL IS FINISHED
9. Press report button
10. Press save, located on the lower left corner of the monitor


## Export Exam to PACS

Pres Patient button, press study list, select patient, press export

## Editing Emergency Patient Information

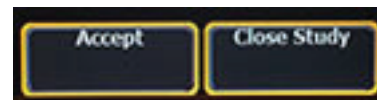

- 1.
  - 2.
- You can edit the information input in the patient data screen:

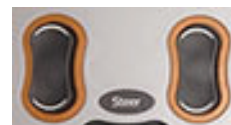

4.

1. Press Patient button
2. Press Edit
3. Enter all the correct information
4. Press Accept

Information on the patient exam images will be updated